

CB501 / CB502 INSTRUCTIONS FOR USE



1 > Place filter into the filter housing.



2 > Turn filter slightly clockwise to lock.



3 > Remove cap to install tubing and replace at end of procedure.



4 > Place hose into the back of the filter housing.



5 > Place other end of hose into Control Panel.



6 > Turn ON/OFF switch to "ON" position, green light will appear.



TESTING FILTER

Recommended testing: Test filter prior to start of each list.



Filter must be installed in housing, and cap removed during testing.



Hosing must be connected to panel and back of filter box, as shown in 4 and 5. System must be on.



Press test filter button, if filter needs replacing, red light will appear.



T +613 9550 1819
M + 61 409 865 105
E Info@Endovision.com.au
W Endovision.com.au