


# CB503 IP INSTRUCTIONS FOR USE



1 > Place filter into the filter housing.



2 > Turn filter slightly clockwise to lock.

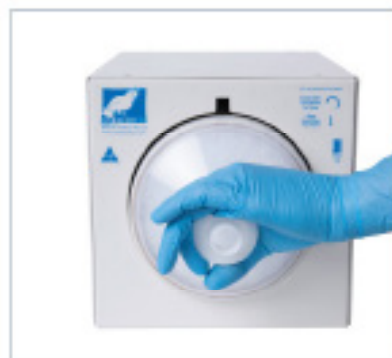

3 > Remove cap to install tubing and replace at end of procedure.



4 > Turn On/Off Switch to On position. Green light will appear.



## TESTING FILTER

Recommended testing: Test filter prior to start of each list.



Filter must be installed in housing, and cap removed during testing.



If filter needs changing red light will appear.



To test Filter. Press "Test Filter" button whilst system in "ON". Test filter at start of each procedure



**T** +613 9550 1819  
**M** + 61 409 865 105  
**E** [Info@Endovision.com.au](mailto:Info@Endovision.com.au)  
**W** [Endovision.com.au](http://Endovision.com.au)